

ANCHOR^{*}

Tissue Retrieval System™



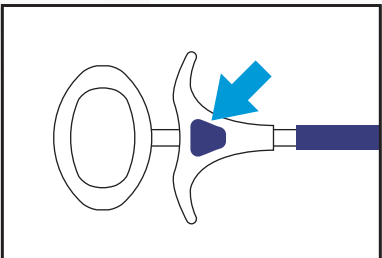
3 ANCHOR^{*} Bag Alone Options

TRS070 Bag Alone		Bag Volume: 175ml Bag Size: 2.2in (5.6cm) x 5.5in (14cm) Trocars Size: 10mm Box Qty: 10
TRS080 Bag Alone		Bag Volume: 250ml Bag Size: 3.5in (8.9cm) x 5.5in (14cm) Trocars Size: 10mm Box Qty: 10
TRS090 Bag Alone		Bag Volume: 1200ml Bag Size: 5.5in (14cm) x 9.9in (25.4cm) Trocars Size: 15mm Box Qty: 5

For final deployment, simply **PUSH-PUSH-PULL**:


Step 1

PUSH and hold the purple button.




Step 2

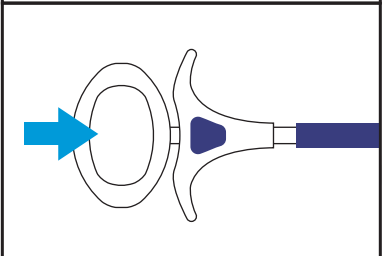PUSH the plunger into the introducer until a 'click' is heard.











Step 3

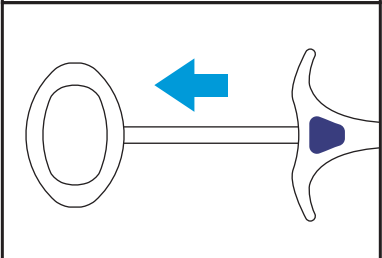PULL back on the plunger to remove it from the introducer tube, allowing for final specimen removal.



7 ANCHOR^{*} Bag on Introducer Options

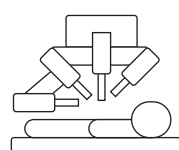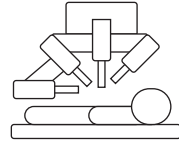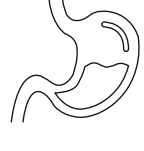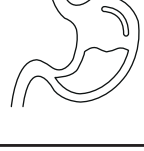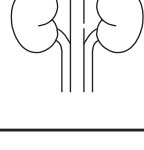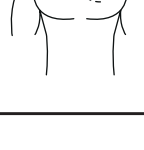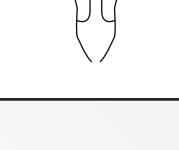TRS-ROBO-8 		Bag Volume: 125ml Bag Size: 2in (5.1cm) x 5.5in (14cm) Trocars Size: 8mm Box Qty: 5
TRS-ROBO-12 		Bag Volume: 250ml Bag Size: 3in (7.6cm) x 5.5in (14cm) Trocars Size: 12mm Box Qty: 5
TRS100SB2 		Bag Volume: 175ml Bag Size: 2.2in (5.6cm) x 5.5in (14cm) Trocars Size: 10mm Box Qty: 5
TRS175SB2 		Bag Volume: 1200ml Bag Size: 5.5in (14cm) x 9.9in (25.4cm) Trocars Size: 15mm Box Qty: 3
TRS190SB2 		Bag Volume: 1800ml Bag Size: 6in (15.2cm) x 10.4in (26.5cm) Trocars Size: 15mm Box Qty: 3
TRS-VATS-15 		Bag Volume: 1200ml Bag Size: 5.5in (14cm) x 9.9in (25.4cm) Trocars Size: 15mm Box Qty: 3
TRS-TV-25 		Bag Volume: 6000ml Bag Size: 6.5in (16.6cm) x 15in (38.1cm) Introducer Size: 25mm Box Qty: 3

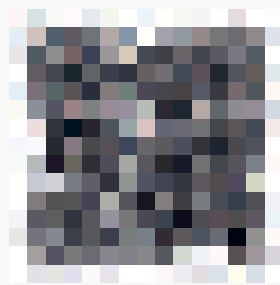

Ask for the “purple one”

Contact your local Sales Representative