

Stiegmann-Goff Ligator™

First in Band Ligation



Attach Friction Fit Adapter
Carefully twist or rock adaptor while pushing it onto distal end of scope.



Insert Trip Wire
Insert ball end of trip wire through biopsy channel until it exits scope.



Position Inner Cylinder
Position inner cylinder onto trip wire with edge closest to "O" ring facing outward.



Secure Trip Wire
Snap trip wire into first groove of inner cylinder.



Line-Up Trip Wire
Slide wire over to snap into the second groove, then line wire up with biopsy channel.



Complete Assembly
Fit the inner cylinder into the adaptor. The ligator is now ready to use.

Stiegmann-Goff™ Endoscopic Ligator Kit

Non-Sterile, Disposable

Catalog Number	Description	Inner Cylinders	Adapter/Cylinder Color	Maximum Scope Length	Scope Tip Diameter	Minimum Working Channel O.D.	Unit
200220	Endoscopic Ligator	5	Gray	130cm	9-13mm	2.8mm	5 kits/box
200221	Endoscopic Ligator	10	Gray	130cm	9-13mm	2.8mm	5 kits/box
000221	Clearvue™ Endoscopic Ligator	5	Clear	130cm	9-13mm	2.8mm	5 kits/box
000227	Clearvue™ Endoscopic Ligator	10	Clear	130cm	9-13mm	2.8mm	5 kits/box

Prior to use please consult product labels and inserts for any indications, contraindications, warnings, precautions, adverse reactions, and instructions for use.

Each Stiegmann-Goff™ Endoscopic Ligator Includes:

- 1 ea. Monofilament trip wire
- 1 ea. Friction fit adaptors for 9-11mm and 11-13mm scopes
- 5 ea. (200220/000221) inner cylinders with preloaded bands or
- 10 ea. (200221/000227) inner cylinders with preloaded bands

Stiegmann-Goff™ Bandito™ Endoscopic Ligator

Non-Sterile, Disposable

Catalog Number	Description	Endoscope Compatibility	Adapter/Cylinder Color	Maximum Scope Length	Scope Tip Diameter	Minimum Working Channel	Unit
100225	Endoscopic Hemorrhoid Ligator	Colonoscope	Clear	210cm	13-15mm	2.8mm	5 kits/box
100226	Endoscopic Hemorrhoid Ligator	Flexible Sigmoidoscope	Clear	80cm	13-15mm	2.8mm	5 kits/box

Prior to use please consult product labels and inserts for any indications, contraindications, warnings, precautions, adverse reactions, and instructions for use.

Each Stiegmann-Goff™ Bandito™ Ligator Kit Includes:

- 1 ea. Monofilament trip wire
- 1 ea. Friction fit adaptor for 13-15mm scopes
- 3 ea. Clear inner cylinder with preloaded bands

Prior to use please consult product inserts for indications, contraindications, instructions for use, warnings and precautions